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(EN) Fertility Gel – Instructions for Use (Medical Device)

Dear Customer,

Thank you for choosing our Fertility Gel! These instructions will help you to use the product in an optimal way. Please read the instructions for use – in particular the "safety instructions" carefully, before the initial application of the Fertility Gel! We would be glad to respond

to your queries or suggestions!

Your ELANEE Team

Please read the instructions for use carefully before using the Fertility Gel. Do not discard the instructions so you can consult them again if necessary.

1. Product contents

- 6 tubes of Fertility Gel, 5 ml each
- 1 instructions for use

2. Intended use/Indications

ELANEE Fertility Gel supports the motility of sperm as they reach the egg, thus increasing the probability of fertilization. In addition, its content is effective in maintaining their viability, as it simulates the environment in which sperm can survive.

Mechanism of action:

The Fertility Gel is formulated to provide a pH balance to increase the survival of the sperm. It contains magnesium and calcium ions to support conception.

3. Application

Fertility Gel is applied inside the vagina before sexual intercourse.

1. Remove the tube from the sachet. Break off the tip of the tube by bending the thin part sideways.
2. Place the tip of the tube inside the vagina as shown and gently squeeze the bottom of the tube.

4. Safety instructions

Contraindications:

- Infection in the genital area
- Irritated genital area
- Known allergic reaction to one of the ingredients
- Breastfeeding
- Excessive vaginal secretion

Side Effects:

When used as directed and safety instructions are followed, no side effects are known.

Warnings:

- Fertility Gel is not a contraceptive.
- Do not use as a vaginal lubricant.
- Avoid contact with eyes.
- Do not swallow.
- Use the entire contents of the tube; do not store for a second application.
- Perform general personal hygiene after application and after intercourse.
- Keep out of reach of children.
- Do not use if allergic to the ingredients.
- Do not use after curettage.

5. Storage, shelf life and handling

Store between 5 °C and 30 °C. Protect from direct sunlight. Store in a dry and cool place.

6. Disposal Instructions

Please separate the individual packaging components and dispose of them in the respective recycling system. Please observe regional disposal regulations.

7. Ingredients

Aqua, Hypromellose, Methyl Paraben Sodium, Mono Sodium Phosphate Anhydrous, Sodium Chloride, Potassium Chloride, Sodium dihydrogen phosphate monohydrate, Magnesium Chloride, Calcium Chloride, Glycerin, Sodium Hydroxide, Citric Acid

8. Explanation of symbols used

	Observe the instructions for use
	Medical device
	Batch number
	Manufacturer
	Use by
	Distributor
	Temperature limits

	Unique Device Identifier
	Catalog number
	Protect from sunlight
	Non-sterile
	Attention!
	Protect from moisture
	Single use only
	Do not use if the packaging is damaged
	Date of Manufacture (Month/Year)
	Model Number

Date of preparation of the instructions for use: 2025-11-07

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