ELANEE®

Gebrauchsanweisung Soft-Tampons

(DE)	Soft-Tampons – Gebrauchsanweisung.....	2
(EN)	Soft tampons – Instructions for use.....	9
(FR)	Soft-Tampons – Noticed'utilisation.....	16
(IT)	Tamponi soffice – Istruzioni per l'uso.....	23
(NL)	Soft Tampons – Gebruiksaanwijzing.....	30

LOT siehe Verpackung und Einzelverpackung / see the packaging and the individual package / voir l'emballage et l'emballage individuel / si veda la confezione e l'involucro singolo / zie verpakking en afzonderlijke verpakking

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EN Soft tampons - Instructions for use

Dear Customer,

Thank you for choosing our Soft tampons! The instructions for use should help you to get the best use out of your product. Please read the instruction manual carefully - particularly the "Usage and safety instructions" - before using this product for the first time. We will be glad to answer any questions and to receive any suggestions you might have!

Your ELANEE team

 Please observe and retain the instructions for use and the packaging!

1. Field of application

These instructions for use are applicable for the following product:

REF 742-00 Soft tampons

2. Product content

- 4/10/50 soft tampons, individually packaged
- 1 instructions for use

3. Purpose and intended use

The Soft tampon is designed for discrete, hygienic and absorbent protection during menstruation. The soft tampon has standard absorbency and is therefore ideal for use throughout the period, depending on how heavy the period is. The soft tampon is designed exclusively

for insertion into the vagina during the period.

4. Areas of application

Because they don't have threads, the tampon can be worn in everyday situations as well as for sport, in the sauna, when swimming or during sexual intercourse. Soft tampons can be used when at home as well as when you are out and about.

5. Functionality

The Soft tampon is inserted into the vagina to just below the cervix. Due to the absorbent material, the liquid is absorbed straight away and leaks are prevented. After insertion, the tampon adapts to the shape of the body, thanks to its ergonomic shape and therefore prevents the liquid from leaking. The tampon should be worn for a maximum of 8 hours. Depending on how heavy the period is, it may need to be changed earlier. The soft tampon can be removed by pulling it out or hooking a finger in the tab.

6. Usage

Before inserting:

- Wash your hands thoroughly before using the soft tampon.
- Remove the tampon from the packaging.
- In order to make it easier to insert the tampon, you can moisten it with clean water. Then squeeze it properly, to guarantee the highest level of absorbency.

Insertion:

1. Hold the soft tampon on the outside between two fingers and place a third finger on the flat side of the soft tampon.
2. Now press it firmly together.

Tip: In order to make it easier to insert, it helps to have a relaxed posture. For example, the tampon can be inserted when standing (possibly with one leg lifted) or when squatting.

3. With your other hand, open the labia and push the tampon into the vagina up to the cervix.

How long to wear it: It is important that you change the tampon regularly. The soft tampon can be worn for up to 8 hours. But it may also be necessary to remove the tampon before that, depending on how heavy your period is. We recommend that you wear it for 4-6 hours. After sexual intercourse or swimming you should change the soft tampon immediately.

Removal: The soft tampon does not have a removal string, it can be removed simply using the integrated tab. To do this hold the tab using the index or middle finger and pull it out. If it is difficult to remove, we recommend squatting or raising one of your legs. If you are not able to remove your soft tampon, please consult your gynaecologist immediately.

7. Safety instructions

 The soft tampon is a single-use product and must not be used more than once in order to prevent infection.

 Ensure a clean environment when using the soft tampon (e.g. wash your hands).

 Check your soft tampon before using it. Only use products which are sealed and in perfect condition. Do not use a soft tampon if the packaging is damaged.

 If an over-sensitivity or allergy to the material is known, the product should not be used or only used after consultation with a physician.

 Please do not use the product if you have any inflammation, injuries, open wounds or painful conditions in your genital area, so as not to interfere with the healing process.

 It is important that you change the tampon regularly. Before inserting a new tampon, please make sure you have removed the previous one.

 If you keep the tampon inserted in your vagina for more than 8 hours, then safety can no longer be guaranteed and it may no longer be effective.

 After sexual intercourse or swimming you should change the soft tampon immediately in order to prevent infection.

 Please always remember to remove the last tampon at the end of your period.

⚠ If you are using a coil or vaginal ring as contraceptive, please discuss using a soft tampon with your doctor.

⚠ The soft tampon is not a contraceptive and does not provide any protection against sexually transmitted diseases.

⚠ If while using or after using a soft tampon you experience itching or notice a rash, remove the soft tampon immediately and consult your gynaecologist.

⚠ If you are not able to remove the soft tampon yourself, please consult your gynaecologist.

⚠ Insert the tampon carefully. Improper handling of the tampon can lead to injury to the hymen or vaginal mucosa.

Important information about Toxic Shock Syndrome (TSS)

TSS is a rare but serious and sometimes fatal illness, which can occur when using tampons.

Symptoms of TSS: The symptoms can appear suddenly and are similar to those of flu. These include sudden fever (39 °C or above) as well as vomiting, diarrhoea, fainting, dizziness or a skin rash.

Actions to take for suspected TSS: Should you experience one of these symptoms during or shortly after menstruation, remove the tampon immediately and consult a doctor straight away. Early detection and treatment is very important and can prevent serious

illness. It is essential that you tell your doctor that you are having your period and have used a tampon.

Risk groups: TSS can occur in all menstruating women using tampons. Women under 30 have an increased risk. If you have any questions about TSS please contact your gynaecologist.

8. Safekeeping, service life, storage and transport

The soft tampon should be stored in its original packaging in a dry place, away from light. Soft tampons are everyday consumables which are designed for single use only. The soft tampon must be disposed of after use.

9. Cleaning and disposal instructions

Used soft tampons should be disposed of in a sanitary bag or a tissue in household waste, in order to prevent the risk of infection. Please be aware of regional disposal guidelines. Used soft tampons must not be disposed of down the toilet. The individual components of the packaging can be disposed of as waste paper.

10. Material composition

The Soft tampon consists of 100% polyurethane foam.

11. Explanations of symbols used

Manufacturer

Single use only

Batch number

Article number

Observe the instructions for use

Attention

Keep dry

Protect from sunlight

Do not use if the packaging is damaged

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