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EN Vaginal gel – Instructions for use (Medical device)

Dear Customer,

Thank you for choosing our ELANEE Vaginal gel! The instructions for use should help you to get the best use out of your product. Please read the instruction manual carefully – particularly the usage and safety instructions – before using this product for the first time. We will be glad to answer any questions and to receive any suggestions you might have!

Your ELANEE-Team

Before using the product, please read the instructions for use, as they contain important information for the patient regarding the use of the product. Keep the instructions for use so that you can read them again if necessary.

REF 739-V1 Vaginal gel

1. Product content

- 7 applicators with vaginal gel
- 1 Instruction for use

2. Purpose and intended use

The ELANEE Vaginal gel (medical device) is a high-quality gel that forms a protective layer on the surface of the mucous membrane, which moisturises and protects the vaginal mucosa from epithelial damage caused by vaginal dryness, during intercourse, gynaecological examinations or physical activity. The use of the vaginal gel improves vaginal elasticity, maintains and restores the physiological pH of the vagina, which helps to maintain the normal bacterial flora in the vagina and reduces unpleasant odours.

The vaginal gel has an acidic pH and helps restore the natural, physiological environment of the vagina, which translates into maintaining the proper microbiome of the entire female reproductive system (the vaginal microbiome is the entire set of microorganisms, including bacteria, that live in the host and regulate a number of physiological functions). Therefore, the medical device can be used to treat bacterial vaginosis (BV) and as a supplement to therapies used in the prevention and treatment of endometriosis.

Under normal conditions, the pH of the vagina is around 3.8 - 4.5, which promotes the growth of beneficial bacteria such as Lactobacillus, which inhibit the growth of pathogenic bacteria such as:

- Gardnerella vaginalis
- Streptococcus sp., Enterococcus sp., E. coli, Enterobacter, Trichomona
- fungi of the genus Candida

The vaginal gel can be safely used by non-professional users at home.

The product is intended for women of childbearing age, as well as those in and after menopause.

Indications

- Prevention of adverse changes in the vaginal flora and pH (e.g. travel, swimming pool, sauna, fitness, antibiotic therapy) by moisturising and maintaining the natural pH of the vagina
- Relief of symptoms of vaginal dryness (discomfort, tension, itching, burning of the vaginal mucosa) associated with:
 - Atrophy of the vaginal mucosa
 - Oestrogen deficiency
 - Dyspareunia (painful intercourse)
 - Prolonged stress
- For additional lubrication of the vaginal epithelium:
 - After surgical and gynaecological procedures, after abrasions and other minor mechanical injuries to the vaginal mucosa
 - After childbirth
 - After chemotherapy, radiotherapy, cryotherapy
 - After laser procedures
 - In the case of changes caused by the use of hormonal drugs
 - During perimenopause
 - For additional vaginal lubrication, which helps reduce discomfort during intercourse
- To reduce unpleasant vaginal odour associated with abnormal vaginal pH due to disturbances in the natural vaginal microflora,
- For the treatment of bacterial vaginosis (BV)
- To supplement therapies used in the treatment and prevention of endometriosis. The product is easy and hygienic to apply, ensuring a high level of comfort. It is odourless, non-hormonal and hypoallergenic. It does not contain PEG, parabens or detergents, including: SLS, SLES.

Mode of action

The product acts at the site of application through a physical mechanism.

After vaginal application, a moisturising protective layer is formed on the vaginal mucosa, which protects the vaginal epithelium from mechanical damage and reduces discomfort associated with dryness, tension, itching and burning of the vaginal mucosa.

The protective layer has an acidic pH, which contributes to the normalisation of the vaginal pH and the microbiome of the entire female reproductive system.

3. Usage

Wash your hands thoroughly before applying the gel!

1. Remove the applicator cap.
2. Insert the applicator tip into the vagina.
3. Squeeze the entire contents of the applicator into the vagina.
4. Remove the applicator.
5. Discard the used applicator. Do not reuse the same applicator!

Use the product immediately after opening the applicator.

Unless otherwise advised by your doctor, use the contents of one applicator per night for 7 consecutive days. Then, to maintain the effect of the product, it is recommended to use one applicator every 2-3 days until the symptoms subside. Do not use the product for more than 30 days.

If your doctor prescribes a different method of use than the one described above, follow your doctor's instructions.

If you miss a dose, do not use a double dose of gel. If symptoms persist, consult your doctor or pharmacist about further use of the product.

You can also use 1 dose of the product immediately before intercourse.

The product is intended for vaginal use only.

The product is intended for use in disposable applicators and contains a single dose for one-time use.

If the entire dose of the product is not used, the applicator should be disposed of. Do not attempt to reuse an opened product at a later date. This is inconsistent with the guidelines for use and storage of the product and may carry a risk of infection at the application site due to loss of product stability due to inconsistent storage conditions.

4. Safety instructions and contraindications

- Hypersensitivity to the product ingredients / occurrence of adverse reactions after use of the product:
 - Do not use the product if you are hypersensitive (allergic) to any of its ingredients.
 - If the patient has no known hypersensitivity to the ingredients of the product and during use of the gel a burning sensation, itching or redness appears which does not subside spontaneously, this may indicate hypersensitivity to the ingredients of the product, in which case you should discontinue use of the product and consult your doctor or pharmacist.
 - When using the product in patients with severe vaginal dryness, due to the lactic acid content of the product, slight reactions may occur at the site of application (mild burning or itching). These symptoms are temporary and usually disappear spontaneously. If the symptoms persist, this may indicate hypersensitivity to the ingredients of the product. In this case, discontinue use of the product and consult your doctor or pharmacist.
- During pregnancy and breastfeeding, use only after consulting your doctor or pharmacist.
- Do not use when trying to conceive, as the pH of the gel has an adverse effect on sperm.
- Do not use during menstruation.
- If vaginal dryness persists, consult your doctor or pharmacist.
- Avoid accidental contact of the gel with the mucous membranes of the eyes, nose or mouth.
- Do not use with local anaesthetics and other vaginal preparations, including vaginal contraceptive preparations, as the product may weaken their effect.
- Do not use the product if the applicator is damaged.
- Do not reuse an applicator once it has been opened.
- Do not use after the expiry date.
- If the entire dose of the product has not been used, the applicator should be disposed of. Do not attempt to reuse an opened product at a later date. This is contrary to the guidelines for use and storage of the product and may carry a risk of infection at the application site due to loss of product stability as a result of improper storage conditions.
- Use the product immediately after dispensing the dose from the packaging, i.e. from the tube. The dispensed product cannot be used at a later time.
- The tube is a single-use dose; any unused amount should be disposed of in accordance with applicable regulations.
- The product is intended for vaginal use only. Do not swallow!

Adverse reactions and serious incidents related to the use of the medical device should be reported to the manufacturer or distributor and to the competent state authority.

5. Safekeeping, service life, storage and transport

Store at temperatures between 15 °C and 25 °C in the original packaging. Protect from light and heat sources. Keep out of the reach and sight of children. Do not use after the expiry date stated on the packaging. Do not use if the packaging is damaged.

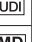
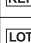
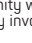
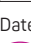
6. Disposal instructions

Please separate the individual packaging components and dispose of them in the appropriate recycling system. Please observe the regional disposal guidelines. The product does not contain any environmentally harmful substances.

7. Ingredients

1 applicator contains: sodium hyaluronate, lactic acid, glycerol, propylene glycol, hydroxyethylcellulose, phenoxyethanol, ethylhexylglycerin, purified water. Ingredients responsible for the essential action of the product: Sodium hyaluronate (0.01%), Hydroxyethylcellulose (1.7%), Glycerol 85% (4.0%), Propylene glycol (3.0%), Purified water (90.29%)

8. Explanation of symbols used

	Manufacturer		Best before date
	Observe the instructions for use		Unique Device Identifier
	Article number		Medical device
	Batch number		Distributor
	Protect from sunlight		Non-sterile
	Temperature limits		Single use only
	Marking indicating the product's conformity with the MDR, together with the identification number of the notified body involved in the conformity assessment of the product.		
	Do not use if the packaging is damaged		

Date of preparation of the instructions for use: Edition C dated 2025-11-01

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