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## **(EN)** Vaginal-Ovules – Instructions for use (Medical device)

Dear Customer,

Thank you for choosing our ELANEE Vaginal-Ovules! The instructions for use should help you to get the best use out of your product. Please read the instruction manual carefully – particularly the usage and safety instructions – before using this product for the first time. We will be glad to answer any questions and to receive any suggestions you might have!  
Your ELANEE-Team

Before using the product, please read the instructions for use, as they contain important information for the patient regarding the use of the product. Keep the instructions for use so that you can read them again if necessary.

**(REF)** 738-V1 Vaginal-Ovules

### 1. Product content

- 10 Vaginal-Ovules
- 1 Instruction for use

Weight of one ovule: approx. 1.6 g

### 2. Purpose and intended use

The ELANEE Vaginal-Ovules (medical device) creates a protective barrier on the vaginal mucosa, moisturizing it and protecting it from damage caused by vaginal dryness, intercourse, gynecological examinations, or physical activity. Using the Vaginal-Ovules improves vaginal elasticity, maintains and restores physiological vaginal pH, which promotes healthy vaginal flora and reduces unpleasant odor. After application, the medical device forms a protective layer with an acidic pH, which helps to restore the natural, physiological environment of the vagina, thus maintaining the normal microbiome of the entire female reproductive system (the vaginal microbiome is the entire set of microorganisms, including bacteria, that occur naturally in the vagina and regulate a number of physiological functions). Therefore, the product can be used to treat bacterial vaginosis (BV) and as a supplement to therapies used in the prevention and treatment of endometriosis.

Under normal conditions, the pH of the vagina is around 3.8-4.5, which promotes the growth of beneficial bacteria such as Lactobacillus, which inhibit the growth of pathogenic bacteria such as:

- Gardnerella vaginalis
- Streptococcus sp., Enterococcus sp., E. coli, Enterobacter, Trichomona
- Candida fungi

The product can be safely used by non-professional users at home.

The product is intended for women of childbearing age, as well as those in and after menopause.

### Indications

- Prevention of adverse changes in the vaginal flora and pH (e.g. travel, swimming pool, sauna, fitness, antibiotic therapy) by moisturising and maintaining the natural pH of the vagina
- Relief of symptoms of vaginal dryness (discomfort, tension, itching, burning of the vaginal mucosa) associated with:
  - Atrophy of the vaginal mucosa
  - Oestrogen deficiency
  - Dyspareunia (painful intercourse)
  - Prolonged stress
- For additional lubrication of the vaginal epithelium:
  - After surgical and gynaecological procedures, after abrasions and other minor mechanical injuries to the vaginal mucosa
  - After childbirth
  - After chemotherapy, radiotherapy, cryotherapy
  - After laser procedures
  - In the case of changes caused by the use of hormonal drugs
    - During perimenopause
- To reduce unpleasant vaginal odour associated with abnormal vaginal pH due to disturbances in the natural vaginal microflora,
- In the treatment of bacterial vaginosis (BV),
- To supplement therapies used in the treatment and prevention of endometriosis

Non-hormonal product

Does not contain PEG, parabens, detergents, including: SLS, SLES

### Mode of action

The product acts at the site of application through a physical mechanism. After applying vaginally, a moisturising protective layer is formed on the mucous membrane, which protects the vaginal epithelium from mechanical damage and reduces discomfort associated with dryness, tension, itching and burning of the vaginal mucosa.

The protective layer has an acidic pH, which helps to normalise the pH of the vagina and the microbiome of the entire female reproductive system. Lactic acid and glycogen are responsible for the acidic pH of the product at the application site.

### 3. Usage

1. Wash your hands thoroughly before applying the product.
  2. Remove 1 ovule from the blister pack by tearing the foil along the perforation and opening the foil wings.
  3. Insert the ovule deep into the vagina.
- Note: If the ovule is too soft, place it in the refrigerator for a few minutes before removing it from the foil until it hardens.
- Unless your doctor advises otherwise, use 1 ovule per night for 7 consecutive days. Then, to maintain the effect, take 1 ovule every 2-3 days until the symptoms subside. Do not use the product for more than 30 days.
- If your doctor recommends a different method of use than the one described above, follow your doctor's instructions.
- If you miss a dose, do not use a double dose.
- This product is for vaginal use only!

The product is intended for single-dose, single-use as one ovule. Insert the ovule immediately after removing it from the blister pack. An ovule removed from the blister pack cannot be used at a later time. This is inconsistent with the guidelines for use and storage of the product and may carry a risk of infection at the application site due to loss of product stability due to inconsistent storage conditions.

### 4. Safety instructions and contraindications

- Hypersensitivity to the product components / occurrence of adverse reactions after use of the product:
  - Do not use the product if you are hypersensitive (allergic) to any of its ingredients.
  - If the patient has no known hypersensitivity to the ingredients of the product and during use of the gel a burning sensation, itching or redness appears which does not subside spontaneously, this may indicate hypersensitivity to the ingredients of the product, in which case you should discontinue use of the product and consult your doctor or pharmacist.
  - When using the product in patients with severe vaginal dryness, due to the lactic acid content of the product, slight reactions may occur at the site of application (mild burning or itching). These symptoms are temporary and usually disappear spontaneously. If the symptoms persist, this may indicate hypersensitivity to the ingredients of the product. In this case, discontinue use of the product and consult your doctor or pharmacist.
- Do not use with local anaesthetics or other vaginal preparations, as the product may weaken or delay their action.
- Do not use with latex condoms and contraceptive inserts, as the product may reduce their effectiveness due to its fat content.
- Do not have sexual intercourse immediately after applying the ovule.
- During pregnancy and breastfeeding, use after consulting your doctor or pharmacist.
- In case of acute vaginal infection, use only after consulting your doctor.
- The product can be used with other vaginal products if recommended by a doctor.
- Do not use during menstruation.
- Avoid accidental contact of the product with the mucous membranes of the eyes, nose or mouth.
- The product is intended for vaginal use only.
- Do not use after the expiry date.
- Do not use the product if the blister pack is damaged.
- Do not reuse an ovule once it has been inserted.
- If the entire dose of the product is not used, the product should be disposed of. Do not attempt to reuse an opened product at a later date. This is inconsistent with the guidelines for use and storage of the product and may pose a risk of infection at the application site due to loss of product stability as a result of improper storage conditions.

Adverse reactions and serious incidents related to the use of the medical device should be reported to the manufacturer or distributor and to the competent state authority.

### 5. Safekeeping, service life, storage and transport

Store at temperatures between 15 °C and 25 °C in the original packaging. Protect from light and heat sources. Keep out of the reach and sight of children. Do not use after the expiry date stated on the packaging. Do not use if the packaging is damaged.

### 6. Disposal instructions

Please separate the individual packaging components and dispose of them in the appropriate recycling system. Please observe the regional disposal guidelines. The product does not contain any environmentally harmful substances.

### 7. Ingredients

1 vaginal ovule contains: sodium hyaluronate, glycogen, lactic acid, tocopheryl acetate, sodium lactate, Witepsol S58  
Ingredients responsible for the essential effect of the product: sodium hyaluronate – 10 mg/ovule

### 8. Explanation of symbols used

	Manufacturer		Best before date
	Observe the instructions for use		Unique Device Identifier
	Article number		Medical device
	Batch number		Distributor
	Protect from sunlight		Non-sterile
	Temperature limits		Single use only
	Marking indicating the product's conformity with the MDR, together with the identification number of the notified body involved in the conformity assessment of the product.		
	Do not use if the packaging is damaged		

Date of preparation of the instructions for use: Edition C dated 2025-11-01

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