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Handwäsche

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nicht bügeln

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Instructions for using ELANEE support belt for pregnant women

Dear customer,

Thank you for choosing our ELANEE support belt! These instructions will help you in using our product with the best possible results. Please read the instructions for use carefully - in particular the „safety advice” - before using this product for the first time.

Please feel free to contact us in case of any questions or suggestions. We hope you will be pleased with our support belt!

Your ELANEE team

Please observe and retain the instructions for use and the packaging!

Made in Europe

1. Field of application

These instructions for use apply to the following products:

REF 751-00 ELANEE support belt for pregnant women, size S

REF 752-00 ELANEE support belt for pregnant women, size M

REF 753-00 ELANEE support belt for pregnant women, size L

REF 754-00 ELANEE support belt for pregnant women, size XL

2. Product content

- 1 ELANEE support belt for pregnant women in the selected size (S, M, L, or XL)
- 1 instructions for use

3. Purpose and intended use

In the course of pregnancy and with increasing weight of the child, the belly of the expectant mother is pulled down more and more. This causes additional load on the backbone. The consequences are frequent back pain and a slight pull in the abdomen.

The ELANEE support belt for pregnant women supports the belly and mitigates back pain by relieving the back and pelvic base muscles.

After childbirth, it can be used for stabilising and strengthening the abdominal wall.

4. Functionality

The continuously adjustable Velcro® fastener makes it possible to adjust the belt as per individual needs. Due to this, the supporting belt can be used during the entire course of pregnancy and after childbirth. With two additional elastic bands, the supporting effect can be adjusted for comfort and stability based on personal requirements. The breathable material allows the skin to breathe providing for a pleasant experience.

5. Application

It is important that you place the ELANEE support belt correctly in order to achieve an optimum supporting effect. The product images on the packaging show you the

correct positioning of the belt. Please observe the following steps in the process:

1. **Open** the support belt by loosening the elastic bands from the Velcro ® fastener, which are fastened on the back of the belt.
2. **Place** the belt on your back with the large contact surface at the spinal disc and close the belt **below the belly** with the **right end above the left end**. The **arrow** on the label should point **upwards**.
3. **Seal** the Velcro ® fastener as snugly as it is comfortable for you.
4. Take **both elastic bands** into your hands simultaneously and tighten them as **evenly** as possible, so that there are no folds in the fabric that could cause a pressure sensation.
5. **Fasten** the elastic bands on the intended Velcro ® fastener.
6. If you feel that the supporting belt is too taut or loose after you stand up, repeat step 1 to 5 and vary the tension in the process.

Tip: The belt is best worn while **lying on the back**.

6. Safety information

The supporting belt should only be used under load and not during rest periods.

7. Cleaning and care instructions

The ELANEE support belt is only suitable for hand wash and should be dried in open air without wringing or twisting. To prevent damage to the support belt, the use of chemical cleaning agents and ironing the support belt is not recommended. Please observe the cleaning and care instructions on the product in order to increase the service life of your belt.

8. Disposal information

The ELANEE support belt can be disposed of along with the household waste. Please observe the local and regional disposal regulations in the process.

9. Material composition

The ELANEE support belts are composed of:

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