ELANEE®

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Instructions for use ELANEE breast milk bag

Dear customer,

Thank you for purchasing the ELANEE breast milk bag. We are glad that you have chosen our product. In order to make optimal usage of the ELANEE breast milk bag possible, we have compiled important information in these instructions for use. Please read it thoroughly. We are happy to help, if you have any questions or suggestions!

Your ELANEE team

1. Scope

These instructions for use apply to the following product:

REF 710-00 ELANEE breast milk bag

2. Product contents

- 20 breast milk bags
- 1 instructions for use

3. Purpose and intended use

The ELANEE breast milk bags make hygienic and safe storage and freezing of pumped breast milk possible. You can freeze and store up to 180 ml of breast milk in one ELANEE breast milk bag. It can also be used for storing and transporting baby food.

4. Field of applications

The ELANEE breast milk bags are suited for filling, storing and transporting breast milk (or baby food). The bags are intended for storing pumped milk at room temperature, for cooling in the refrigerator and for freezing in the freezer. It is essential that you heed the information about breast milk shelf life on the

back of the packaging or in these instructions for use under item 7!

5. Application

Storing and freezing breast milk:

1. Make sure that everything is clean when handling breast milk (e.g. wash hands, clean jars).
2. Take the bag from the storage bag only just before using it.
3. Open the safety closure.
4. Fill the pumped milk into the bag.
5. Read off the filling quantity from the scale.
6. Press all the air out of the bag and close the bag.
7. Enter the filling date and the quantity into the intended labelling field, preferably with a waterproof pen.
8. Place the bag flat in refrigeration or freezer.
9. Make sure during storage that the bag does not come in contact with other frozen items or that it is pulled across icy surfaces. Otherwise, the material could become brittle and porous. Frozen breast milk bags should be stored very carefully, preferably in an extra compartment.

Defrosting breast milk:

1. Defrost the filled milk bag slowly in the refrigerator or at room temperature.
2. Put the bag upside down into a clean jar. This way, possibly leaking milk can be collected. If a clean jar is used, the collected milk can be used as food without hesitation.
3. For quick defrosting, the breast milk can be defrosted under running warm water or in a water bath (max. 37°C).
4. If the milk forms different layers, shake the bag, so that milk

and milk fat are mixed again.

5. Do not freeze defrosted milk again.

6. Safety information

 Use breast milk bag only once.

 Do not freeze defrosted milk.

 Do not mix warm milk with frozen milk.

 For defrosting, do not use boiling water, a microwave or an oven!

7. Information about shelf life and storage

Observe the breast milk shelf life according to the following temperature information:

- 16-29 °C up to 4 hours
- Below 4°C up to 3 days
- Below -18 °C up to 6 months

8. Disposal information

The used ELANEE breast milk bags can be disposed of in the household waste (residual waste). Please separate the individual components of the packaging correctly sorted and transport them to the appropriate recycling system such as Yellow Bag or paper container.

9. Material composition

The ELANEE breast milk bags are composed of 100 % polyethylene.

ELANEE breast milk bags are:

- free of bisphenol A (BPA)
- pre-sterilised

- freezer-safe
- food-safe
- neutral in taste
- extra resistant
- furnished with a double safety closure
- provided with a clear labelling field
- provided with an indication of measurements

10. Explanations of the symbols used

Manufacturer

Observe instructions for use

Caution!

Only for single-use

Batch number

Item number

Please observe instructions for use and packaging and keep them!

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