ELANEE®

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Achtung!

Trocken lagern

Instructions for use ELANEE Breast-Rings

Dear Customer,

Thank you for purchasing our ELANEE Breast Rings. These instructions for use shall help you to use the product optimally.

Please read the instructions for use carefully- in particular the „safety advice“ - before using this product for the first time.

We will be glad to answer any questions and receive any suggestions you might have.

Your ELANEE-Team

Read and keep instructions for use and packaging!

Made in Germany

1. Area of Application

These instructions for use apply to the following product:

REF 222-00 ELANEE Breast-Rings

2. Contents

- 5 Pairs of ELANEE Breast-Rings in a polybag (5 pieces with and 5 pieces without a notch)
- 1 Instructions for Use

3. Purpose and Intended Use

ELANEE Breast-Rings protect sensitive, painful, sore and inflamed nipples from external contact and contact with the bra due to their hollow shape. This helps soothe pain and promotes healing through air circulation.

The ELANEE Breast Rings prevent the bending or squeezing of the nipples, they improve the blood circulation of the nipples and promote the healing process.

The breast rings are not suitable for any other purposes than the ones named above.

4. Area of Application

ELANEE breast rings are particularly suited for breast-feeding women with sensitive, painful, sore or inflamed nipples.

5. How it works

The rubbing of clothing material or contact with the bra encourage nipple problems and can be responsible for painful nipples. The breast rings are highly elastic with a strong compression hardness and create a barrier between the breast and the textile materials. The special pear shape of the breast rings protects the breast optimally and contributes to an even distribution of pressure.

A breathable hollow support for the nipples is recommended to protect them and to encourage a fast healing process.

The circular opening of the breast rings ensures the access of air and at the same time relieves painful nipples from external touch or contact. This promotes regeneration and improves the healing process of sore nipples.

The breast rings are provided with or without a notch to differentiate their positioning on the right and left breast. This prevents cross-contamination when reused after feeding.

6. Application

1. Ensure clean conditions prior to the application by washing and drying your hands.
2. Loosen your bra (adjust straps and fastening generously).
3. Take a breast ring with a notch from the packaging for your left breast and place it into the centre of the left-hand cup of your bra with the notch pointing downwards.

Take a breast ring without a notch and place it onto the right breast with the broader side pointing downwards (“pear-shaped”).

4. During the milk let-down, the breast rings should not be worn longer than 20 minutes and thereafter 30 minutes, without checking the breasts.

If there are any signs that the mammary ducts are compressed, the bra needs to be loosened further. If this does not help, stop wearing the breast rings.

5. Remove breast rings prior to feeding.
6. After feeding, place the ring with the notch onto the left-hand breast and the one without the notch onto the right-hand one.
7. Dispose of contaminated or damp rings at once in your household waste - at the latest however after 24 hours and apply a new breast ring as described under point 2 and 3.

Tip: If one ring is not sufficient to surround the nipple in the hollow, use a second ring and place it on top of the first one.

7. Safety Advice

-  Ensure a clean environment (e.g. wash your hands)!
-  Please check your breast rings prior to using them for the first time. Only use wrapped breast rings in perfect condition. Do not use already open or damaged wrapping.
-  The breast rings must not be pressed too hard against the breast (loosen up your bra), as otherwise the mammary ducts could be compressed and the milk flow interrupted). This can create a stasis oedema.
-  Side effects could be skin irritation such as erythema, swelling, burning or itching of the breast. In case of a known hypersensitivity and allergic reactions to one of the mentioned material components, you should stop using the breast rings or only use them after having consulted your physician.

-  In case of infected nipples, third-degree burns, heavily exudating wounds, bleeding nipples or mycosis use only after having consulted your physician.
-  Painful nipples may also be a sign of a malignant disease. Therefore, in case of sore, painful or inflamed nipples, breast rings should only be used in addition to therapeutic medical measures.
-  Please consult your doctor, if the symptoms get worse or do not improve.
-  The breast rings are designed to be used only once and up to a maximum of 24 hours . Dispose of contaminated or damp rings and replace with new ones in order to avoid any risk of infection.

8. Storage and Shelf Life

Please keep breast rings dry - ideally in the unopened polybag.

9. Advice on Cleaning and Disposal

The breast rings are intended to be used only once and up to a maximum of 24 hours.

The breast rings can be disposed of in the usual household waste.

Separate the individual components of the packaging accordingly and dispose of in the relevant recycling system.

Please observe the local and regional guidelines for disposal.

10. Material Composition

The ELANEE-Breast Rings consist of 100% Polyurethane foam.

10. Explanation of symbols used

	Manufacturer		Read Instructions for Use
	Batch Number		Single use only
	Article Number		Attention!
			Store in a dry place

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