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---

## Instructions for use ELANEE Hydrogel Pads

### Dear Customer,

Thank you very much for choosing our ELANEE Hydrogel Pads. These instructions for use will help you get the best from this product. Please read these instructions carefully before using the product for the first time.

We will be pleased to answer any further questions you may have.

*Your ELANEE-Team*

Please read the instructions for use and the packaging and keep them in a safe place.

### 1. Area of Application

These instructions for use refer to the following product:

**REF** 30001 ELANEE Hydrogel Pads

## **2. Contents**

- 4 ELANEE Hydrogel Pads, individually wrapped in aluminium pouches
- 1 Instructions for Use

## **3. Purpose and Intended Use**

The ELANEE Hydrogel Pad is a sterile lightly adhesive gel pad, which soothes pain caused by lesions from breastfeeding, dry and/or cracked nipples. The gel pad contains a moist wound environment that promotes the physiological healing process and keeps the skin smooth. The gel pad generates a light, pleasant cooling effect, which is very welcome for breastfeeding women. The Hydrogel Pad does not stick to the skin and can be removed easily for breastfeeding.

The ELANEE Hydrogel Pads are used

- to ease pain caused by cracked or sore nipples;
- to protect sensitive nipples and
- to avoid direct contact with clothing and also
- to keep the area around the nipples moist and smooth to encourage a faster healing process.

The gel pads are not suitable for purposes other than the ones described above.

## **4. How it works**

ELANEE Hydrogel Pads encourage the healing process of sore and cracked nipples and have a pleasant cooling and soothing effect. The pads form a protective layer between nipples and clothing and thus protect sensitive nipples. The cool gel soo-

thes immediately and accelerates the healing process through the principle of healing aided by moisture. Hydrogel Pads do not stick to the wound, can be removed painlessly and do not damage the new skin layer when the pads are changed.

Hydrogel Pads create a moist wound environment, which encourages superficial wounds to heal optimally. Hydrogel Pads form a barrier against microorganisms and thus minimise the danger of the wound becoming infected by bacteria.

## 5. Application

1. Ensure a clean environment prior to the application by washing and drying your hands and subsequently your nipples thoroughly.
2. Only remove the pads from the foil wrapping shortly before use. Tear open the foil bag and pull off the protective foil from the Hydrogel Pad.
3. Place the pad with the gel side directly centrally onto the nipple with the tip pointing upwards.
4. Remove the pad before breastfeeding by gently pulling the tip of the drop-shaped pad off your breast.
5. Gently wash your breast with warm water.

**Tip:** In order to achieve an additional cooling effect, place the Hydrogel Pad in the fridge before using it.

## 6. Safety Advice

 The product is only intended for single use as it is no longer sterile thereafter! If required it can be used several times (e.g. to cool affected nipples without any open

wounds) for up to a maximum of 24 hours! Dispose of soiled pads and replace with new ones.

⚠ Ensure a clean environment (e.g. wash hands and breast) and make sure the Hydrogel Pads are in perfect condition before using them!

⚠ Do not use if sterile wrapping is damaged or after the expiry date!

⚠ Do not use in case of infected nipples, third-degree burns or seriously exudating wounds!

⚠ Only use pads on bleeding nipples or mycotic infections after having consulted your doctor!

⚠ Please consult your doctor in case the symptoms become worse or do not improve!

## **7. Advice on Cleaning and Disposal**

For cleaning, observe the application and safety advice.

The gel pads can be disposed of with the normal household waste.

Separate the individual parts of the packaging according to material and dispose of them in the correct recycling system.

Observe the local and regional guidelines for disposal.

## **8. Material Composition**

- Carrier Material: Polyester fleece
- Polyurethane Film
- Polyurethane-Gel with a 60 % water content

## 9. Explanation of Symbols used

Manufacturer

Attention!

Observe Instructions for Use

Article Number

Batch Number

Do not expose to direct sunlight

Store in dry conditions

Store between 5 °C and 25 °C only

Single use only

Sterilised by irradiation

To be used by

Do not use, if the package is damaged or in case the sterility has been compromised

Does not contain any natural rubber

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