ELANEE®

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### **(GB) Silver Breast Pads - Instructions for use**

#### **Dear Customer,**

Thank you for choosing our silver breast pads! The instructions for use should help you to get the best use out of your product. Please read the instruction manual carefully - particularly the "Usage instructions" - before using this product for the first time. You can download the Instructions for Use from our website, at [www.elanee.de](http://www.elanee.de). We will be glad to answer any questions and to receive any suggestions you might have!

*Your ELANEE-Team*

Please read the instructions for use and the packaging and keep them in a safe place.

Made in Germany

## **1. Field of application**

These instructions for use refer to the following product:

**REF** 218-00 ELANEE Silver Breast Pads

## **2. Product content**

- 2 ELANEE Silver Breast Pads
- 1 Instructions for Use

## **3. Purpose and intended use**

During breast feeding breast nipples may become sore and dry. The exsudate and the leaking milk are absorbed by the nursing pads to keep the breast dry. The silver nursing pads have a cool feeling to them when applied to the skin and permit air to circulate. They are soft and do not irritate the sensitive skin.

## **4. Functionality**

The damp warm atmosphere within the nursing pads encourages the formation of bacteria that will then attack the breast. Thus the skin becomes irritated, the

breasts may become inflamed and cause a mastitis. Also, encouraged by a warm temperature the bacteria may develop a disagreeable, rancid smell. Due to its antibacterial activity, silver (CAS-Nr. 7440-22-4), an active biocide agent, reduces colonization and repopulation of microorganisms in the breast pads and prevents odors. Wool and Lyocell are natural yarns that absorb and store liquid extremely well, which means that breasts and clothing remain dry.

## **5. Usage**

Place the side of the nursing pads containing the silver yarn onto the breasts. You will recognise the correct side by its grey melange and the fine yarn thread. Use the ELANEE nursing pads just as you would use any other standard pads between each breast feeding.

## **6. Cleaning and disposal instructions**

Please soak in lukewarm water with a small amount of detergent, rinse well, absorb water with towels, pull into shape and dry flat. Adding a bit of vinegar to the water helps prevent matting. A certain degree of odor is normal for wool.

The Silver Breast Pads can be disposed of with the normal household waste.

Separate the individual parts of the packaging according to material and dispose of them in the correct recycling system. Observe the local and regional guidelines for disposal.

## 7. Material composition

The ELANEE Silver Breast Pads consist of:

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