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### **GB** Menstrual Cup – Instructions for use

Dear Customer,

Thank you for choosing our menstrual cup! The instructions for use should help you to get the best use out of your product. Please read the instruction manual carefully - particularly the "Usage and safety instructions" - before using this product for the first time. You can download the instructions for use from our website, at [www.elanee.de](http://www.elanee.de). We will be glad to answer any questions and to receive any suggestions you might have!

Your *ELANEE* team

Please observe and retain the instructions for use and the packaging!

#### **1. Field of application**

These instructions for use are applicable for the following products:

740-00 ELANEE Menstrual Cup, size 1

741-00 ELANEE Menstrual Cup, size 2

## 2. Product content

- 1 ELANEE Menstrual Cup in the selected size (1 or 2)
- 1 storage bag
- 1 instructions for use

## 3. Purpose and intended use

The ELANEE Menstrual Cup offers safe and discrete protection during menstruation and is a reusable alternative to tampons and pads.

## 4. Areas of application

Because it is so comfortable to wear, the ELANEE Menstrual Cup can be used during the day as well as when sleeping and also for sports, when swimming or going to the sauna. The special feature of the ELANEE Menstrual Cup: it can be removed easily and hygienically using the unique pull tab or, alternatively, it can be removed like you would remove a standard cup.

The ELANEE Menstrual Cup is suitable for use by women from the first to the very last period and is available in two sizes:

**Size 1:** light to normal periods

Diameter: 40 mm, capacity: 23 ml

**Size 2:** heavy periods

Diameter: 45 mm, capacity: 33 ml

## 5. Functionality

The ELANEE Menstrual Cup is inserted into the vagina to col-

lect the menstrual secretion. The cup, which is folded to insert, then unfolds in the vagina and, thanks to the flexible material it sits against the vaginal wall in such a way that the menstrual blood cannot flow past. Removal is made easier by a special mechanism: the heart-shaped grip of the pull tab, which is connected to the edge of the cup, needs to be pulled. Through the tensile force, the upper edge of the cup releases from the vaginal wall so that the cup can be easily pulled out of the vagina. Alternatively, the ELANEE Menstrual Cup can also be removed just like standard menstrual cups.

## 6. Usage

⚠ Before each use, check the menstrual cup for damage.

### How do you insert the ELANEE Menstrual Cup?

Wash your hands thoroughly before inserting the menstrual cup.

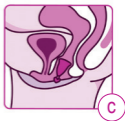
- Fold the cup either into a C-shape (a) or a mussel-shape (b) and when doing this, make sure that the side with the pull tab is pushed towards the middle. Hold the folded cup firmly with your thumb on one side and index and middle finger on the other side as low down as possible.


- Try out different positions to see which will be the most comfortable for you to insert the cup: standing or squatting, with one leg raised

or sitting on the toilet. Relax your vaginal muscles and part your labia with your free hand. Insert the folded menstrual cup into the vagina in the direction of your coccyx until the grip area is approximately 2 cm above your vaginal opening. The menstrual cup will open before it is fully inserted and will sit at the lower end of the vagina (c).

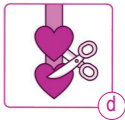- In order to check whether the menstrual cup is in the correct position, hold the cup by the lower part (not on the grip) and rotate the cup half a turn in any direction. If the menstrual cup can be rotated easily, this tells you that it has fully opened.

If you get the feeling that the pull tab is too long, or is annoying, you can cut off one of the hearts (d).

### How do you remove the ELANEE Menstrual Cup?

Depending on how heavy your period is, you should empty the menstrual cup several times a day to make sure it doesn't overflow. We recommend removing the cup and cleaning it after a maximum of 8 hours.

Wash your hands thoroughly and make sure you are in a comfortable position before removing the menstrual cup.

#### Option 1

- Relax the vaginal muscles, hold onto one

heart on the pull tab with your thumb and index finger and pull carefully (e).

- Pull the cup downwards using the pull tab until it is positioned loosely in your vagina.
- Let go of the heart and hold the cup by the grip area to pull it out of your vagina.

## **Option 2**

You can also remove the ELANEE Menstrual Cup just like any standard menstrual cup:

- Relax your vaginal muscles and gently pull on the grip area until you can get hold of the lower part of the cup.
- Press it together and pull the cup out of your vagina.

Alternatively you can slide two fingers between the vaginal wall and the menstrual cup and then pull it out.

Use the markings on the inside of the cup to read off the volume:

- Size 1: 10 ml and 20 ml
- Size 2: 15 ml and 25 ml

Empty the contents into the toilet. Clean the menstrual cup as described in Point 9 and re-insert it.

## **What should you do if you feel uncomfortable when you are wearing the ELANEE Menstrual Cup or when it is leaking?**

First try to remove the menstrual cup as described above and put it back in, just to be sure that the cup is in the correct

position. If it doesn't feel comfortable, then this may be because the selected cup size is not the right one for you. Choosing the right size depends on how heavy your period is, your age, your pelvic floor muscles or whether you have given birth with a vaginal delivery. If you get the feeling that the pull tab is too long, or is annoying, you can cut off one of the hearts (d).

## **7. Safety instructions**

-  When inserting and removing the menstrual cup make sure you observe hygiene conditions (e.g. washing your hands).
-  If you feel pain or experience an unpleasant sensation when you are wearing the ELANEE Menstrual Cup, check whether it is in the correct position and remove it if the symptoms persist.
-  Do not wear the menstrual cup for longer than 8 hours in one go.
-  The ELANEE Menstrual Cup is not to be used as a contraceptive.
-  If you are using an intrauterine device (coil) or a vaginal ring for contraception, consult your gynaecologist before using the ELANEE Menstrual Cup.
-  Remove the ELANEE Menstrual Cup before sexual intercourse.
-  The ELANEE Menstrual Cup does not provide any protection against sexually transmitted diseases.

-  Using the ELANEE Menstrual Cup can possibly lead to damage to the hymen.
-  Only use the ELANEE Menstrual Cup following a birth, operation, miscarriage or a termination after consultation with your gynaecologist.
-  Do not use the ELANEE Menstrual Cup if you are suffering from a vaginal infection.
-  If an over-sensitivity or allergies to the material named is known, the product should not be used or only used after consultation with your gynaecologist.
-  If you are not able to remove the menstrual cup yourself, please consult your gynaecologist.

### **Important information about Toxic Shock Syndrome (TSS)**

TSS is a rare but serious and sometimes fatal illness, which can occur when using menstrual cups.

**Symptoms of TSS:** The symptoms can appear suddenly and are similar to those of flu. These include sudden fever (39 °C or above) as well as vomiting, diarrhoea, fainting, dizziness or a skin rash.

**Actions to take for suspected TSS:** Should you experience one of these symptoms during or shortly after menstruation, remove the menstrual cup immediately and consult a doctor straight away. Early detection and treatment is very important and can prevent serious illness. It is essential that you tell

your doctor that you are having your period and have used a menstrual cup.

**Risk groups:** TSS can occur in all menstruating women using a menstrual cup. Women under 30 have an increased risk. If you have any questions about TSS please contact your gynaecologist.

## **8. Safekeeping, service life, storage and transport**

To store and transport the cup, place the clean, dry menstrual cup into the cotton bag provided.

We recommend replacing the menstrual cup with a new one after a maximum of 5 years, if you use it normally and follow the cleaning instructions as specified.

## **9. Cleaning and disposal instructions**

**Before first use and after every period:** Boil the ELANEE Menstrual Cup in a pot for 3 minutes, immersed in water.

**During your period:** After emptying it, clean the menstrual cup thoroughly with mild soap and rinse it under running water. If it is not possible to rinse the cup (e.g. when in a public toilet), then you can wipe it out with toilet paper or a tissue and clean it properly the next time you have the opportunity.

The menstrual cup can be disposed of along with normal household waste. Please separate the individual packaging components and dispose of them in the relevant recycling system. Please be aware of regional disposal guidelines.

## 10. Material composition

100 % medically-approved TPE

The ELANEE Menstrual Cup has not been tested on animals and does not contain any animal-based ingredients.

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