ELANEE®

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GB Lactic acid treatment – Instructions for Use

Dear customer,
Thank you for choosing our ELANEE lactic acid treatment! These instructions will help you to use the product in an optimal way. Please read the instructions for use – in particular the "safety instructions" carefully, before the initial application of ELANEE lactic acid treatment!

We would be glad to respond to your queries or suggestions!

Your ELANEE-Team

⚠ Please follow the instructions for use and packaging and keep them safely!

Made in Germany

1. Scope

These instructions for use are applicable for the following product:

REF 730-00 ELANEE lactic acid treatment

2. Product contents

- 1 lactic acid gel in 20 ml tube
- 7 separately packed applicators with 2.5 ml filling volume
- 1 instructions for use

There may be a slight or more intense burning sensation once you have started the application due to the low pH-value (pH 3.8) of the lactic acid treatment leading to an increased sensitivity of the vagina. This should disappear as the vaginal flora is restored.

3. Purpose and intended use

ELANEE lactic acid treatment increases the degree of acid in the vagina area safely and effectively. It helps to restore and maintain the natural pH value in the vagina. This supports a well-balanced and healthy vaginal flora. ELANEE lactic acid treatment is also recommended as a prophylaxis for reoccurring bacterial vaginal infections (vaginosis). The treatment is designed for daily application over 7

days. The product is not suitable for any other application. In case of any doubt, please ask your physician or pharmacist!

4. Areas of application

ELANEE lactic acid treatment is particularly suitable for women with a known vaginal imbalance (vaginal pH value > 4.4), with a tendency to vaginal infections (bacterial vaginosis) and following a treatment of a vaginal infection with antibiotics or subsequent to a vaginal mycosis. Restoring a well-balanced and healthy vaginal flora prevents of the symptoms vaginal infections such as itching, burning, abnormal discharge or unpleasant smell.

5. Functionality

Lactic acid bacteria ensure a healthy vaginal flora in the vagina and prevent accumulation and propagation of harmful bacteria. If the count of lactic acid bacteria in the vaginal area is too low, it will lead to an increase in the vaginal pH value and bacteria can settle and propagate faster.

Bacterial vaginal infections are accompanied by many complaints such as itching, burning, dryness, increased vaginal discharge or uncomfortable odour. Severe and untreated infections may lead to inflammations and to infertility. Increasing infections may trigger early labour, ruptured membranes and premature birth in case of pregnant women.

ELANEE lactic acid treatment has a pH value of below 4.0 and allows the vaginal flora to normalise. In addition, the body has sufficient time to form and generate its own lactic acid to regenerate the healthy vaginal flora.

6. Usage

1. Ensure clean conditions (e.g. wash your hands).
2. Unscrew the gel tube and pierce the seal of the tube with the cap pulled upside down.
3. Remove the applicator from the packaging shortly before use and screw the applicator into the gel tube.
4. Squeeze the lactic acid gel **slowly** out of the tube until the stopping point on the screw-on applicator (approx. 2.5 ml = **1/2 an applicator filling**). **WARNING!** Do not push any gel past the plunger. **Carefully** unscrew the half-full applicator from the tube.

Position yourself in supine position, preferably in the evening before going to sleep, with legs pulled in a little towards the body and a hand towel below the pelvic floor area.


Press the gel completely from the applicator into the vagina by **slowly** pressing the plunger. In case of sensitive mucous membranes the gel may cause a slight short-term burning feeling due to its low pH value.

Dispose the used applicator into household waste and repeat the above steps with a new applicator in the following days.

Tipp: It is recommended to use a slip insert/sanitary towel after using the lactic acid gel, because it may lead to increased vaginal discharge.

11. Explanation of symbols used

	Manufacturer
	Attention
	Observe Instructions for Use
	Article number
	Batch number
	Weight
	No direct sunlight
	Default storage temperatures
	Marking of the packaging material
	Only for single use
	To be used by: year and month
	Once opened to be used within 1 month

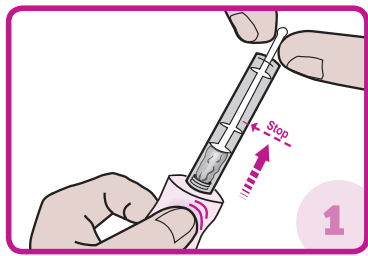Picture 1:
Remove the applicator from its wrapping just before use and screw it onto the opened gel tube. Squeeze the lactic acid gel out of the tube right up to the stop of the screwed-on applicator.

Picture 2:
Insert the filled applicator about 2 - 3 cm into the vagina, just like a tampon. Squeeze the gel into the vagina by **slowly** pressing the applicator's plunger; dispose of the used applicator.

7. Safety instructions

- ⚠ Ensure clean conditions (e.g. wash your hands).
- ⚠ Check tube and applicator prior to using them for the first time. Only use sealed and perfect parts. Do not use already open or damaged tube and applicators.
- ⚠ The applicators are designed for one-off use and must not be re-used in order to avoid the risk of an infection.
- ⚠ In case some gel that is not required for the

current application escapes from the tube unexpectedly, do not put this back into the tube or use for a further treatment - make sure to dispose of the relevant quantity of gel.

Do not use condoms, latex membranes, spermicidal or other types of physical contraceptives while using the lactic acid gel (risk of rupture or ineffectiveness).

The treatment should not be carried out in case of an inflammation, injury or other painful conditions, nor during menstruation.

If an over-sensitivity or allergy to one of the mentioned ingredients is known, treatment should not be carried out or should be carried out only after having consulted a physician.

In case of virginity or during a problematic pregnancy, treatment should only be started after having consulted a general practitioner.

Side effects can include skin reactions such as erythema, swelling, burning sensation or itching at the area of application. A general practitioner should be consulted if your well-being is affected.

Consult your doctor should the symptoms become worse or do not improve within 7 days.

8. Storage and shelf life

Keep the lactic acid treatment out of the reach of babies/toddlers.

The aluminium tube must be kept away from direct sunlight and stored and transported within a temperature range of between 5°C and 25°C. The lactic acid gel has a minimum shelf life indicated on the packaging. The date refers to the last day of the specified month. We strongly advise against using the gel after the expiry date.

The gel must be used within 1 month of opening the gel tube.

9. Disposal instructions

Dispose of all product parts in your household out of the reach of babies or toddlers.

Dispose of and recycle the individual parts of the packaging according to their type. When doing this please note the local and regional waste disposal regulations.

10. Material composition

ELANEE lactic acid gel contains: aqua, glycérine, xanthan gum, glyceryl caprylate, lactic acid

An applicator is made of: Polyethylene (PE)

11. Explanation of symbols used

	Manufacturer
	Attention
	Observe Instructions for Use
	Article number
	Batch number
	Weight
	No direct sunlight
	Default storage temperatures
	Marking of the packaging material
	Only for single use
	To be used by: year and month
	Once opened to be used within 1 month

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