ELANEE®

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GB Breast Rings – Instructions for use

Dear Customer,

Thank you for choosing our ELANEE Breast Rings! The instructions for use should help you to get the best use out of your product. Please read the instruction manual carefully - particularly the usage and safety instructions - before using this product for the first time. We will be glad to answer any questions and to receive any suggestions you might have!

Your *ELANEE-Team*

 Please follow the instructions and retain the instruction sheet and the packaging!

Made in Germany

1. Field of application

These instructions for use apply to the following products:

REF 222-00 ELANEE Breast Rings

REF 222-V1 ELANEE Breast Rings, hospital pack

2. Product content

ELANEE Breast Rings	5 pairs of ELANEE Breast Rings in a polybag (5 pieces with and 5 pieces without a notch), 1 instructions for use
ELANEE Breast Rings, hospital pack	50 pairs of ELANEE Breast Rings, each in a polybag (50 pieces with and 50 pieces without a notch), 1 instructions for use

3. Purpose and intended Use

The ELANEE Breast Rings can be used for a short period of time on a woman's breast when it is lactating. They protect sensitive, painful, sore and inflamed nipples from external contact due to their hollow shape, which can help soothe pain. The product is designed for personal and professional use. The breast rings are not suitable for purposes other than the listed intended ones.

4. Areas of Application

ELANEE Breast Rings are designed for breast-feeding women with sensitive, painful, sore or inflamed nipples.

5. Functionality

Breast rings have a recess in the centre for the nipple. When placed centrally on the breast inside a loose bra, the rings form a barrier between the breast and the fabric by forming a hollow cup, and this means that the stressed-out nipples are able to heal properly. The material is highly elastic with a strong compression hardness and therefore guarantees the gap to the fabric, even under pressure. The special pear shape of the breast rings also provides ergonomically optimised padding for the breast. The breast rings are provided with or without a notch to differentiate their positioning on the right and left breast. This prevents cross-contamination when reused after feeding.

6. Usage

1. Ensure clean conditions prior to using the breast rings (e.g. by washing your hands).
2. Loosen your bra (adjust straps and fastening generously).
3. Please take a breast ring with a notch from the packaging and place it into the centre of the left-hand cup of your bra with the notch pointing downwards.
Please take a breast ring without a notch and place it onto the right breast with the broader side pointing downwards (“pear-shaped”).
4. Remove the breast rings before feeding.
5. After feeding, use the ring with the notch for the left-hand breast and the ring without the notch for the right-hand one.
6. Dispose of contaminated or damp rings at once - at the latest however after 24 hours - in your household waste and use a new breast ring as described under points 2 and 3.

Tip: If one ring is not sufficient to surround the nipple in the hollow, take a second ring and place it on top of the first one. In order to soothe the stressed-out nipples, we recommend using a nipple balm. For protection against moisture, you can insert an absorbent breast pad between the breast ring and your clothes.

7. Safety Instructions

Ensure a clean environment when using the breast rings (e.g. wash your hands).

Please check your breast rings prior to using them for the first time. Only use wrapped breast rings in perfect condition. Do not use the breast rings if the packaging is damaged.

The breast rings must not be pressed too hard against the breast (bra not too tight), as otherwise the mammary ducts could be compressed and become blocked.

Side effects could include skin irritation such as redness, swelling, burning or itching of the breast.

 If an over-sensitivity and allergy to the material is known, the product should not be used or only used after consultation with a physician.

 The breast rings are designed to be used only once and for a maximum of 24 hours. Dispose of contaminated or damp rings immediately and replace with a new ring in order to avoid any risk of infection.

 Painful nipples may also be a sign of a malignant disease. Therefore, in case of sore, painful or inflamed nipples, breast rings should only be used in addition to therapeutic medical measures.

 If the symptoms persist or deteriorate, please seek advice from a doctor.

 All serious incidents occurring in connection with this product must be reported to the manufacturer and the responsible authorities in the user's country of residence.

8. Safekeeping, service life, storage and transport

The product must be stored in a dry place, protected from direct sunlight.

9. Cleaning and disposal instructions

The breast rings are disposable products and must only be used for a maximum of 24 hours. They can be disposed of along with household waste. The individual packaging components must be separated and then disposed of in the relevant recycling system. Please be aware of local disposal guidelines.

10. Material Composition

The ELANEE Breast Rings are made of 100% polyurethane foam.

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