ELANEE®

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GB Breast gel pads – Instructions for Use

Dear Customer,

Thank you for choosing our ELANEE Breast gel pads! The Instructions for use should help you to get the best use out of your product. Please read the instructions for use carefully – particularly the usage and safety instructions – before using this product for the first time. We will be glad to answer any questions and to receive any suggestions you might have!

Your ELANEE-Team

Please follow the instructions and retain the instruction sheet and the packaging!

1. Field of application

These instructions for use are applicable for the following product:

REF 221-00 ELANEE Breast gel pads

2. Product content

- 6 x 2 ELANEE Breast gel pads
- 1 Instructions for use

3. Purpose and intended use

The ELANEE Breast gel pads are a non-active, mechanical barrier and can be used for a short period of time

on a woman's breast when it is lactating. They are used to protect and cover nipples with surface wounds, where the dermis is not broken. ELANEE Breast gel pads do not stick to the nipples and feel pleasant and cool on the skin. The product is designed for personal and professional use. The ELANEE Breast gel pads are not suitable for purposes other than the listed intended ones.

4. Areas of application

ELANEE Breast gel pads are designed for breastfeeding women with sore nipples.

5. Funtionality

The ELANEE Breast gel pads form a mechanical barrier between the nipple and fabric and thus protect sore nipples from external influences. The breast gel pads do not stick to the skin and feel pleasant and cool on the skin.

6. Usage

1. Ensure clean conditions prior to using the breast gel pads (e.g. by washing your hands).
2. Open the foil bag at the perforation.
3. Please take one breast gel pad out of the packaging and pull off the transparent protective film. Then position the pad with the gel side centrally on the nipple. Follow the same steps for the second pad on the other

breast. If you only need one breast gel pad, you can keep the other one in the packaging for up to 24 hours. Close the packaging tightly and put it in the fridge.

4. Remove the breast gel pads before breastfeeding by pulling gently on the rounded edges on the side. Place the breast gel pads on a clean surface with the gel side facing upwards.
5. Wash the breast carefully with warm water.
6. After breastfeeding, put the pads back onto the same breasts, to avoid any possible cross infection.
7. Dispose of the breast gel pads, at the latest after 24 hours, in your household waste and use a new pair as described under points 3-6. Dispose of contaminated breast gel pads immediately.

Tip: For an additional cool feeling, you can store the breast gel pads in the fridge before use.

7. Safety instructions

-  Please check your breast gel pads before use. Only use breast gel pads which are sealed and in perfect condition. Do not use the breast gel pads if the packaging is damaged.
-  If the skin in the breast area becomes irritated, please stop using the breast gel pads.

- ⚠ If an over-sensitivity or allergy to the material is known, the product should not be used or only used after consultation with a physician.
- ⚠ The breast gel pads are designed to be used only once and for up to a maximum of 24 hours. Dispose of contaminated breast gel pads immediately and replace with new pads in order to avoid any risk of infection.
- ⚠ Only use the breast gel pads with surface wounds.
- ⚠ The breast gel pads are marked with an expiration date (see the bottom of the packaging). The product should not be used after the date has passed.
- ⚠ If the symptoms persist or deteriorate, please seek advice from a doctor.
- ⚠ All serious incidents occurring in connection with this product must be reported to the manufacturer and the responsible authorities in the user's country of residence.

8. Safekeeping, service life, storage and transport

The product must be stored between 0 and 30°C and protected from direct sunlight. The breast gel pads are marked with an expiration date (see the bottom of the packaging). The product should not be used after the date has passed.

9. Cleaning and disposal instructions

The breast gel pads are designed to be used only once and for up to a maximum of 24 hours. Breast gel pads can be disposed of along with household waste. The individual packaging components must be separated and then disposed of in the relevant recycling system. When doing this, follow the regional disposal guidelines.

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