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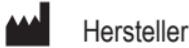
nicht bügeln

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Herstellertdatum (TT.MM.JJ)

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Instructions for using ELANEE support belt

Dear Customer,

Thank you for choosing our support belt! The instructions for use should help you to get the best use out of your product. Please read the instruction manual carefully - particularly the "Usage and safety instructions" - before using this product for the first time. You can download the Instructions for Use from our website, at www.elanee.de. We will be glad to answer any questions and to receive any suggestions you might have!

Your ELANEE team

 Please observe and retain the instructions for use and the packaging!

Made in Europe

1. Field of application

These instructions for use apply to the following products:

- REF** 751-00 ELANEE support belt, size S
- REF** 752-00 ELANEE support belt, size M
- REF** 753-00 ELANEE support belt, size L
- REF** 754-00 ELANEE support belt, size XL

2. Product content

- 1 ELANEE support belt in the selected size (S, M, L, or XL)
- 1 instruction for use

3. Purpose and intended use

The ELANEE support belt is an orthopaedic belt, consisting of a belt with fully adjustable Velcro fastener, elastic straps as well as stabilising back rods and is designed for short-term use in the area of the lower back. It is intended to be worn by women during pregnancy to ease pain in the back or pelvic girdle and after giving birth to stabilise the abdominal wall.

4. Areas of Application

The ELANEE support belt is for expectant mothers with back pain or pelvic girdle pain. The product is only intended for private home use.

5. Functionality

The ELANEE support belt supports the stomach and alleviates back pain by taking some of the burden off the muscles of the back and pelvic floor. The fully adjustable Velcro fastener means that you can adjust the belt to suit individual requirements. This means that the support belt can be worn throughout the entire pregnancy and after giving birth. Using the two additional elastic straps, the supportive effect can be adjusted to fit personal requirements for comfort and stability.

6. Usage Instructions

It is important that you use the right size of ELANEE support belt and that it is positioned correctly, in order to achieve the very best level of support. The graphics and product images on the packaging indicate the correct sizing and the correct positioning of the belt. When doing this, follow these steps:

1. **Open** the support belt by undoing the Velcro fastener.
2. **Place** the belt with the large surface level with the lower back and fasten the belt by putting the **right side over the left side under the stomach**. When doing this the arrow on the label should be pointing **upwards**.
3. **Fasten** the Velcro tight enough that it still feels comfortable.
4. Take **both elastic bands** into your hands simultaneously and tighten them **as evenly as** possible, so that there are no folds in the fabric that could cause a pressure sensation.
5. **Fix** the elastic straps onto the Velcro pads provided, to give you a pleasant supportive effect.
6. If you feel that the support belt is too tight or too loose, repeat steps 1 - 5 and change the tension of the elastic straps.

There is an application video on our website to show you how to position the belt correctly.

7. Safety Instructions

-  The support belt must only be worn when standing or walking and not during breaks or when carrying out sedentary tasks. If, while wearing the belt, you should start to feel unwell (dizziness, high pressure) then please undo the support belt immediately.

-  Side effects could include skin irritation such as redness, swelling, burning or itching due to a build-up of heat in the area of the support belt.
-  The support belt must not be worn if you have open wounds in the stomach, pelvic and back area.
-  Before use, check the support belt to make sure that it is intact. Do not use the support belt if, for example, the rods on the back are sticking out, in order to avoid any risk of injury.
-  The belt will not help with Symphysis Pubis Dysfunction (loosening of the symphysis).
-  If the belt is pressing too hard or slips, then you might need a different size. See the measurement chart/sizing on the packaging.
-  In case of a known hypersensitivity and allergic reactions to one of the material components mentioned, you should stop using the support belt or only use it after having consulted your physician.
-  If the symptoms persist or worsen, please seek advice from a doctor.
-  All serious incidents occurring in connection with this product must be reported to the manufacturer and the responsible authorities in the user's country of residence.

8. Safekeeping, service life, storage and transport

With normal usage, the product should be replaced after 5 years.

9. Cleaning and disposal instructions

The ELANEE support belt is only suitable for hand wash and should be dried in open air without wringing or twisting. To prevent damage to the support belt, the use of chemical cleaning agents and ironing the support belt is not recommended.

Please observe the cleaning and care instructions on the product in order to increase the service life of your belt.

The ELANEE support belt can be disposed of along with the household waste. Please observe the regional disposal regulations in the process.

10. Material composition

The ELANEE support belts are composed of:

- 56 % polyamide (PA)
- 24 % polyurethane (PU)
- 20 % cotton (CO)

10. Explanations of the symbols used

 Manufacturer	 Observe instructions for use
LOT  Batch number	REF  Item number
 Do not dry	 Caution!
 Hand wash	 Do not bleach
 Do not iron	 Do not clean chemically
 Date of manufacture (DD.MM.YY)	MD  Medical Device

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