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Artikelnummer

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---

## **GB** ELANEE Wool Pads – Instructions for use

**Dear customer,**

Thank you for buying this product. Below you will find important product information. Please read this carefully, in particular the safety advice, as well as the information about use, storage and disposal.

We will be glad to answer any questions and to receive any suggestions you might have!

*Your ELANEE team*

Please follow the instructions and retain the instruction sheet and the packaging!

## **1. Scope of application**

These instructions for use apply to the following product:

**REF** 224-00 ELANEE Wool Pads

## **2. Contents**

- 20 ELANEE Wool Pads
- 1 instructions for use

## **3. Intended purpose and use**

The ELANEE Wool Pads are made of carefully selected, gently washed merino sheep wool from controlled biological livestock. The felted and airy structure of the Wool Pads covers stressed nipples and sensitive areas of the skin (e.g. baby bottoms, heels during sports...) discretely and simply without being visible. The Wool Pads with natural lanolin feel pleasantly soft against the skin, are breathable and warming.

## **4. Usage**

The front of the packaging has an opening. Pressing gently here will open the perforated tab. Then you can pull the opening tab forwards and remove the Wool Pads individually. The opening can be folded up again at the top once pads have been removed to protect the remaining Wool Pads. Place the wool pad on your selected area of the skin (e.g. nipple, baby bottom, heel during sports...).

The ELANEE Wool Pads are suitable for use with a light milk flow when breastfeeding. The Wool Pads can simply be placed into a (nursing) bra and worn for as long as they feel comforta-

ble. The felted surface keeps the product securely in place and prevents slipping.

### **Our tip:**

Combine the ELANEE Wool Pads with the ELANEE ultra-thin breast pads – absorbent, very discreet and therefore ideal when out and about. The adhesive point enables the nursing pads to be easily stuck to the bra. You can then place the wool pad on top, with the other side touching the body.

## **5. Usage and safety instructions**

-  Plant remains on the pads are a natural result of the gentle washing process.
  -  Do not use on open wounds or neurodermatitis!
  -  If you use this product on babies, ensure that the Wool Pads are only used under direct adult supervision!
  -  If you notice any redness, irritation or swelling, there is the possibility of a very rare allergy to wool alcohol. Stop using the product immediately.
  -  The Wool Pads are designed to be used only once for hygiene reasons.
- Please do not wash the Wool Pads as this reduces the amount of naturally present lanolin.
  - The Wool Pads are square shapes to prevent valuable wool cuttings being wasted.
  - If you like, you can stretch or cut the pads yourself into the shape you want (e.g. oval).

## 6. Storage and disposal instructions

Store this product in a dry place, and protect against humidity. The Wool Pads are designed to be used only once. Soiled or used pads can be disposed of with household waste. The individual packaging components must be separated and then disposed of in the relevant recycling system. When doing this, follow the regional disposal guidelines.

## 7. Material composition

100% wool (controlled biological livestock)

Natural product

## 8. Explanation of symbols used

 Manufacturer  Observe the instructions for use

 Attention!  Batch number

 Article number  Protect against moisture!

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