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LOT Chargennummer **REF** Artikelnummer

GB Breast milk storage bags – Instructions for use

Dear Customer,

Thank you for choosing our ELANEE Breast milk storage bags! The Instructions for use should help you to get the best use out of your product. Please read the instructions for use carefully – particularly the usage and safety instructions – before using this product for the first time. We will be glad to answer any questions and to receive any suggestions you might have!

Your ELANEE-Team

 Please follow the instructions and retain the instruction sheet and the packaging!

1. Field of application

These instructions for use are applicable for the following product:

REF 711-00 ELANEE Breast milk storage bags

2. Product content

- 20 breast milk storage bags
- 1 instructions for use

3. Purpose and intended use

ELANEE Breast milk storage bags are ideal for safe and hygienic storing and freezing of expressed breast milk.

4. Areas of application

ELANEE Breast milk storage bags are made for filling, storing and transporting breast milk at room temperature, for cooling in the refrigerator or freezing.

5. Functionality

The thermally sealed seams and integrated double safety seal enable safe and hygienic storage, transport and freezing of breast milk without leakage. ELANEE Breast milk storage bags can be stood upright for easier filling. The bags can also be laid flat to save space when storing the breast milk. Plus, the scale printed on the bags shows you exactly how much milk you have expressed so you can write the amount in the space provided on the bag.

6. Usage

Storing and freezing breast milk:

1. Ensure that everything is clean before using the breast milk storage bags and when handling breast milk (e.g. wash your hands, use clean containers).
2. Take the bag from the storage bag only just before using it.
3. Open the safety seal at the perforation.
4. Fill the expressed breast milk into the bag and check the fill volume on the scale printed on the bag.
5. Squeeze all the air out of the bag and close it.
6. Write the date of filling and the amount of milk on the bag in the space provided with a waterproof pen.
7. Lay or stand the bag in the refrigerator or freezer. When storing, ensure that the bag do not come into contact with or freeze onto other items in the freezer. This could damage the bag. Frozen breast milk should be stored in a separate compartment where possible.

Please see the notes under point 8 regarding the shelf life of breast milk.

Defrosting breast milk:

1. Thaw the frozen breast milk storage bag slowly in the refrigerator or at room temperature.
2. Put the bag upside down into a clean jar. This way, possibly leaking milk can be collected. If a clean jar is used, the collected milk can be used as food without hesitation.
3. For quick defrosting, the breast milk can be defrosted under running warm water or in a water bath (max. 37 °C).
4. If the milk separates into layers, shake the bag to mix the milk and milk fat.

Do not refreeze thawed breast milk.

7. Safety instructions

-  For defrosting, do not use boiling water, a microwave or an oven!
-  Please check the breast milk storage bag before use. Only use breast milk storage bags which are in perfect condition. Do not use breast milk storage bags that are damaged or already open.
-  Do not refreeze thawed breast milk.
-  The breast milk storage bags are designed to be used only once and may not be reused.
-  Do not exceed the maximum fill volume of 180 ml.

8. Safekeeping, service life, storage and transport

Observe the breast milk shelf life according to the following

temperature information:

- 16-29 °C up to 4 hours
- Below 4 °C up to 3 days
- Below -18 °C up to 6 months

9. Cleaning and disposal instructions

ELANEE Breast milk storage bags are single-use products and may not be reused. Used breast milk storage bags can be disposed of along with household waste. The individual packaging components must be separated and then disposed of in the relevant recycling system. When doing this, follow the regional disposal guidelines.

10. Material composition

ELANEE Breast milk storage bags are made of:

- 80 % polyethylene (PE)
- 20 % polyethylene terephthalate (PET)

ELANEE Breast milk storage bags are free from bisphenol A (BPA).

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