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EN Menstrual heat patch – Instructions for use

Dear Customer,

Thank you for choosing our ELANEE Menstrual heat patch! The Instructions for use should help you to get the best use out of your product. Please read the instruction manual carefully – particularly the usage and safety instructions – before using this product for the first time. We will be glad to answer any questions and to receive any suggestions you might have!

Your ELANEE-Team

 Please observe and retain the instructions for use and the packaging!

1. Field of application

These instructions for use are applicable for the following product:

REF 734-00 ELANEE Menstrual heat patch

2. Product content

- 2 ELANEE Menstrual heat patches
- 1 instructions for use

3. Purpose and intended use

The ELANEE Menstrual heat patch provides soothing warmth during menstruation. It serves as a warmth dispenser at home and on the way.

4. Areas of application

The ELANEE Menstrual heat patch provides pleasant warmth in the desired area of the body and helps to relax. In addition to

application on the abdomen, the heat patch is also suitable for sticking on the back, shoulders or neck.

The application is not suitable

- for a duration of more than 12 hours within 24 hours.
- for children under 12 years of age.
- on a site with inflammation, bruising, redness, swelling, skin tears, bruising, skin irritation or hypersensitive, diseased, injured or damaged skin.
- together with analgesics and in cases of restricted temperature sensation as well as arterial circulatory disorders and Raynaud's syndrome.
- in combination with medicinal and cosmetic creams, agents and lotions applied to the skin.
- for people who cannot remove the patch on their own.
- if the product is damaged or leaking.
- in case of rheumatic diseases, circulatory problems, diabetes, circulatory disorders, heart diseases, rheumatoid arthritis, nerve damage, diseases of the sensory functions, skin diseases.
- during pregnancy.

5. Functionality

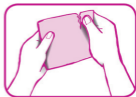
After opening the plastic foil, the filling of the heat patch reacts with oxygen and heats itself. The ELANEE Menstrual heat patch then remains warm for up to 12 hours.

6. Usage

Before use, check if the foil pouch and the heat patch are intact.

Do not use the product if it is damaged.

Step 1: Open the foil pouch at the perforation by hand to remove the heat patch. The heat patch is activated by contact with the air and then heats up by itself - the pouch should therefore only be opened when the heat patch is to be used.

Step 2: Remove the release paper from the adhesive side of the patch.

Step 3: Stick the heat patch on the abdomen above the panties or on the inside of the panties. Place the patch smoothly - it should not wrinkle. The heat patch warms up slowly and reaches the maximum temperature after about 30 minutes, which lasts for at least 12 hours.

Tip: Alternatively, you can also apply the heat patch to your back, shoulders or neck for relaxation.

Wear the heat patch for as long as you find it comfortable. We recommend that you do not wear the patch for more than 12 hours in any 24-hour period.

Remove the heat patch slowly after a maximum of 12 hours. If you notice a slight reddening of the skin, this is due to the warming effect of the patch. If the skin is still red after a few hours, do not use a heat patch until the redness has completely subsided.

The patch is for single use only and should be discarded after use.

7. Safety instructions

- ⚠ Incorrect application can cause burns even at low temperatures. Therefore, monitor the application at regular intervals.
- ⚠ If a slight reddening of the skin occurs after application, this is no cause for alarm. Please allow this to subside before further application.
- ⚠ Ensure that there is air circulation over the heat patch to prevent the risk of burns forming.
- ⚠ To avoid creating pressure on the part of the body to which the patch is applied, do not lie or sit on the product and do not wear tight-fitting clothing over the patch.
- ⚠ Stop using the product immediately, if it becomes uncomfortably hot, a rash develops or you do not feel well.
- ⚠ The product is for external and functional use only.
- ⚠ The product is a single-use product and must not be reused.
- ⚠ In case of accidental ingestion of the contents or contact with the skin or eyes, remove the product, rinse the affected area with water and seek medical attention immediately as a precaution. The contents may be harmful if swallowed or in direct contact with the skin or eyes.
- ⚠ The sensitivity of the skin increases from the age of 55. The heat patch should therefore be worn over thin clothing if it is sensitive.
- ⚠ Keep the product out of the reach of children.
- ⚠ Do not:
 - cut, tear, burn, knead or press the heat patch.

- heat the patch in a microwave or by any other external means.
- let the heat patch come into contact with water or other liquids.

8. Safekeeping, service life, storage and transport

Please store the ELANEE Menstrual heat patch away from children, pets, moisture and direct sunlight in a cool, dry place at room temperature. In the closed foil pouch, the product is stable until the stated expiry date on the back of the foil pouch.

9. Cleaning and disposal instructions

The ELANEE Menstrual heat patch can be disposed of in household waste as soon as it has cooled down completely. Please separate the individual components of the packaging by type and take them to the appropriate recycling system. Please observe the regional disposal guidelines.

10. Material composition

Iron, water, Kieselguhr, activated charcoal, sodium chloride

11. Explanation of symbols used

Manufacturer

Observe the instructions for use!

Attention!

Use by: (Year and Month)

Keep out of direct sunlight

Protect from moisture!

Single use only

Do not use if the packaging is damaged!

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