DE VAGINALER PH-SELBSTTEST

Selbstdiagnosetest. *In-vitro*-Diagnostikum.

TESTPRINZIP

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EN VAGINAL PH SELF TEST

Test for self-diagnosis. *In vitro* diagnostic medical device.

TEST PRINCIPLE

The **Vaginal pH self test** is a reliable, painless method to determine vaginal pH. As soon as the pH measurement area of the applicator comes into contact with vaginal secretions, its colour changes. The colour obtained corresponds to a pH value on the measuring scale and gives the test result. Regular use of the Vaginal pH self test is a reliable way to detect vaginal flora disorders. The strip test range is 3.8-7.0.

INFORMATION

An acidic vaginal pH (\leq 4.5) i s essential for the proper functioning of the vaginal protection system in women of reproductive age. There are multiple factors that regulate the composition and function of the vaginal microbial ecosystem and preserve its natural protective functions. The lactobacilli are one of the main factors. So the most important and natural protection against vaginal issues is having a healthy vaginal flora. Contamination by pathogenic germs and getting vaginal infections causes issues for the vaginal flora and, consequently, a variation in pH. For example, bacterial vaginosis increases the risk of pelvic inflammatory diseases and can also increase the risk of becoming infected with HIV. For pregnant women, it increases the risk of miscarriage, premature birth and so on. One out of every two women with bacterial vaginosis experience no symptoms. That is why, by regularly measuring the vaginal pH of both pregnant and not pregnant women, the risk of any complications is reduced.

Vaginal pH can increase in the event of:

- Bacterial vaginosis
- Sexually transmitted diseases
- Premature rupture of the amniotic sac (waters breaking)

STORAGE

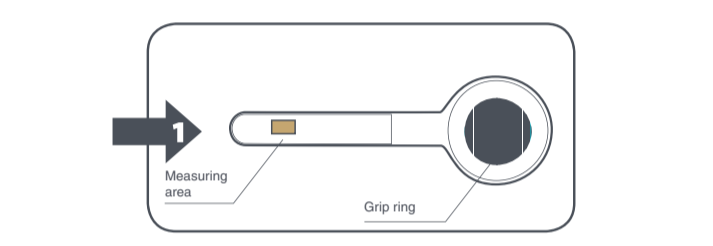
Keep the test at room temperature in the closed cardboard box, in a place out of reach of children. Keep away from direct sunlight and humidity. If you have any questions about this product, please call Customer Service in your country. Our staff are at your disposal.

BOX CONTENTS

- test applicators packed in plastic pouches
- 1 instruction leaflet
- 1 colour scale printed on each plastic pouch

READ OF THE RESULT

Read the result as soon as the applicator is removed from the vagina, while the measuring area is still wet. Compare the colour of the pH measurement area with the colour scale and select the colour closest to your measurement. After use, dispose of the applicator in household waste.

PRECAUTIONS FOR USE

Use each applicator only once. Use this product only for its intended purpose. Do not swallow. The applicator only measures vaginal pH and does not detect infections. An acid pH is not always enough to prevent infections. If you notice any disorders despite a normal pH value, consult your doctor. Certain circumstances may temporarily change vaginal pH and thereby falsify the results of this measurement. Antibiotics may change the bacterial environment of pH vaginal. Thus, a change in the pH might occur and give invalid results. Menopause can induce a decrease in the vaginal bacterial flora, especially lactobacilli. This can result in an increase in pH. Take the following limitations into account when measuring your pH and interpreting the test.

Perform the measurement:

- at least 12 hours after sexual intercourse.
- at least 12 hours after vaginal administration of medicinal products (pessaries, cream, gel, etc.).
- 3 to 4 days after your period ends.
- at least 15 minutes after urinating; traces of urine may lead to a false result.

Do not use right after showering or washing the external genital organs, as soap and cleaners can alter the vaginal pH. Never start a treatment without discussing the test result with your doctor. Incorrect use of the applicator may cause deflowering in virgin women.

LIMITS

Comparing with the colour chart depends on individual interpretation. Therefore, it is recommended that the person reading the results of these applicators has been tested for colour blindness. As with every diagnostic test, a definitive clinical diagnosis must not be based on the results of a single rapid test, but by a doctor after evaluating all clinical and laboratory results.

WHEN TO PERFORM THE TEST?

- In the event of bacterial vaginosis
- After anti-infective treatment (e.g. antibiotics)
- As part of preventive examinations
- During pregnancy
- In case of a history of premature births
- After sexual intercourse with a new partner
- In the event of discomfort in the genital organs (burning, itching, unpleasant smell, discharge)
- Before and after surgery in the vagina

PERFORMANCES

SENSITIVITY AND SPECIFICITY

The ELANEE Vaginal pH self test has been evaluated on buffer solutions with a pH of 3.8 to 7.0. An evaluation by pH electrode served as a reference. The study involved 279 samples.

	%
Sensitivity	100%
Specificity	96,8%

A feasibility study showed that:

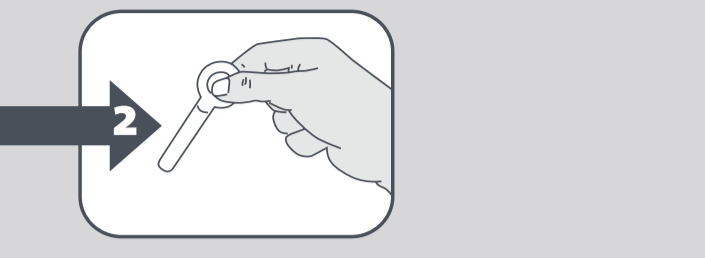
- 100% of women of all ages were able to determine their own vaginal pH
- 96% of women found the correct pH value in pH solutions with the self-test.

DIRECTIONS FOR USE

Use the test before the expiry date (see date on the box).

Measuring vaginal pH is easy and completely painless. Relax and follow the instructions below very precisely.

- Wash your hands before starting the test.
- Open the box, take out a test and open the plastic pouch. Do not open the sachet before to be ready to do the test.
- Hold the applicator by the grip ring between the thumb and index finger, as shown in the illustration below (Figure 2). The thumb must always be on the side of the vaginal applicator, as shown. Do not touch the test measurement area.

- Avoid the pH measuring area coming into contact with other liquids.
- Sit down with the knees apart or stand with one foot on the toilet seat or another stable surface.
- With your free hand, gently free the opening of the vagina by spreading the external labia.
- Gently introduce the applicator into the vagina as if you were using a tampon (Figure 3) up to about 1 to 3 cm deep (no further than the grip ring allows). Inserting the vaginal applicator is completely painless.

- Press the tube gently against the posterior wall of the vagina (on the side of the rectum) for about 10 seconds to ensure that the pH measuring area is well moistened. Sufficient moistening is very important for the test's interpretation.
- Finally, carefully remove the applicator from the vagina.
- Read the result immediately. Do not read after 30 seconds.

INTERPRETATION OF THE RESULT

	Normal vaginal pH (3.8 - 4.4)
You probably have thrush if you also have the following genital disorders:	
• Itching	
• Burning	
• Abnormal discharge resembling "curdled milk"	
Consult your pharmacist, who can recommend a suitable treatment. If the problems persist, consult a doctor.	

	Abnormally high vaginal pH (> 4.4)
You probably have a bacterial infection (vaginosis) if you also have the following clinical disorders:	
• Itching	
• Unpleasant, fishy smell	
• Abnormal liquid white to greyish discharge	
You probably have a Trichomonas if you also have the following genital disorders:	
• Pain when urinating	
• Unpleasant mould-like smell	
• Abnormal foamy yellow to greenish discharge	

Some medicinal products or physiological circumstances may lead to false positive or negative results. If the test shows an abnormal or high result, please perform a new one. If the problem persists, consult a doctor. Never make a medical decision without consulting your doctor.

BIBLIOGRAPHY

1. Redelinghuys MJ, Geldenhuys J, Jung H, Kock MM. Bacterial Vaginosis: Current Diagnostic Avenues and Future Opportunities. Front Cell Infect Microbiol. 2020;10:354. Published 2020 Aug 11. doi:10.3389/fcimb.2020.00354
2. Coudray MS, Madhivanan P. Bacterial vaginosis-A brief synopsis of the literature. Eur J Obstet Gynecol Reprod Biol. 2020;245:143-148. doi:10.1016/j.ejogrb.2019.12.035
3. Lin YP, Chen WC, Cheng CM, Shen CJ. Vaginal pH Value for Clinical Diagnosis and Treatment of Common Vaginitis. Diagnostics (Basel). 2021;11(11):1996. Published 2021 Oct 27. doi:10.3390/diagnostics11111996

	Importer	IVD	In vitro diagnostic medical device
	Consult instructions for use		Do not re-use
	Number of tests per kit		Use-by date
	Temperature limit	LOT	Batch code
REF	Catalogue number		Manufacturer
	Identificaton code for packaging materials		Distributor
	Keep dry	CH REP	Authorized Representative in Switzerland
	Do not use if packaging is damaged and consult instructions for use		Common signage TRIMAN

CH REP	
BIOSYNTAX SWISS SA Route de Rossemaison 100, 2800 Delémont - Switzerland	

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GRÜNSPECHT Naturprodukte GmbH Münchener Str. 21 85123 Karlskron Germany www.elanee.de Tel.: 08450 68 033 0